

# QUANTIFERON – TB PLUS–

## Labeling Tubes - Quick Guide

Effective March 23, 2021

**Affix the label with color designation identical to the color of the blood collection tube.**

**(Color of the tube is indicated on the lower right hand side of the label)**

**Please see the example below of how to label each tube with the correct COLOR LABEL.**

**ERRONEOUS RESULTS WILL OCCUR IF THE WARDE SOFT LABEL INDICATING THE COLOR OF THE TUBE IS NOT MATCHED WITH THE CORRESPONDING COLORED TUBE.**

THIS COLOR NEEDS TO MATCH THE TUBE COLOR.

USE THE PROCESSING LABEL ONLY. PROCESSING LABEL HAS THE BLACK BAR.

Tube Color	Label Color Designation
Yellow	YELLOW
Lavender	LAVENDER
Green	GREEN
Gray	GRAY

Label Content (Common to all):  
10000044091  
TESTPATIENT, TEST2021 46 M 01/01/75  
WX0000003436 [REDACTED] R  
Barcode  
C309000002 03/09/21 13:35 WH11H/WHRL  
;QFT4E  
10.00 Quantiferon \* [COLOR] \*