


# COVS – SARS-Cov-2 Qualitative NAAT, Saliva Instruction for SDNA-1000 Saliva Collection Device

**DO NOT eat, drink, smoke, or chew gum for at least 30 minutes before giving your saliva sample.**

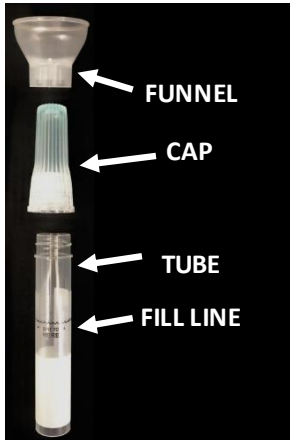

2ml Saliva Collection Device  
Contains Tube, Funnel, Cap and  
Instructions.




**SPECTRUMDNA™**  
A BRAND OF ECHINAP SOLUTIONS, LLC



1. Attach the Funnel.
2. Fill the tube with saliva (not including bubbles) to the black wavy line. **DO NOT OVERFILL.**



4. Replace the funnel with the cap.
5. Firmly screw the cap down to release the stabilizing solution and seal the tube.

*(You will know it works when the blue solution from the cap is released into the tube)*



3. Shake the tube vigorously for at least five seconds. *(This will ensure your sample mixes thoroughly with the stabilizing solution)*

